




EASYROLLER, METAL-FREE WHEELCHAIR

PROCEDURE FOR CLEANING AND DISINFECTION

Golden Rules: “Clean and Disinfect according to procedure at the end of every day as a minimum.
Disinfect with wipes between each patient.”

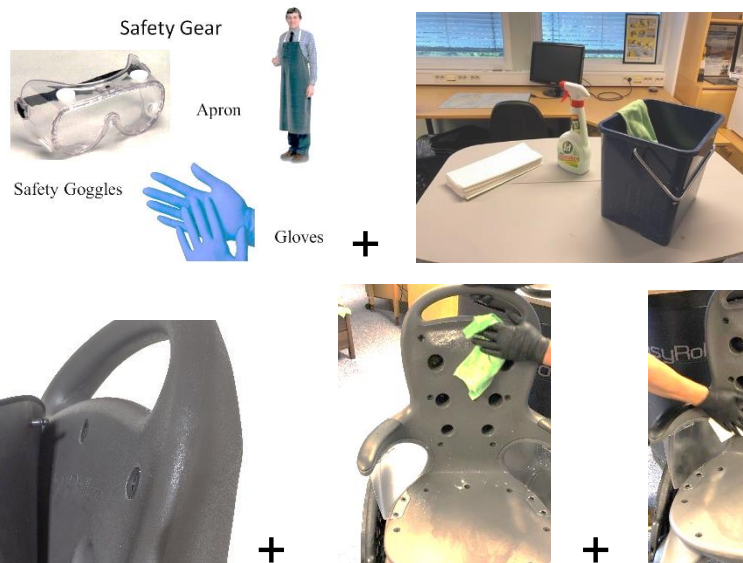

1. Pre-clean - remove excess contamination by, wiping or pre-rinsing – REMEMBER DISMANTLING CUSHIONS prior to cleaning.
2. Main clean – wash off any “body-fluids” using a detergent.
3. Rinse - remove detergent residue – and dry off..

DISINFECTION



Our recommendation is to use “Clinell” or “Distel” wipes for Disinfection.
(These wipes have no negative effect on the mechanical properties of the material in the chair – nor does it create any stains or miscolouring)



Take due care when disinfecting details on the EasyRoller (arrows) – especially the details in touch with people or floor.

Apply disinfectant according to instructions from manufacturer



Skien, Norway 2019


Knut Bernhard Iversen, CEO



EasyRoller